



1 Connect Mixed Gas & Vacuum Scavenger hoses from the J-Box, through the chair and to back of the RFS Block

2 Locate Existing 1/2"-20, Bolt Holes under chair arm.

3 Attach Bracket & Spacers. (1)RFS Bracket (27730), (2) Spacers (27807), (2) 1/2"-20 x 1" Bolts

4 Attach RFS block to bracket with (4) 8-32 x 1.75" Screws (kit 27870)

5 Attach the Remote Bag Tee & Scavenger Controller to the front of the RFS Block

